

Realize Language Starter's Guide

Log in and Explore



Selecting data to explore

When you first log in to Realize Language, you will be in the “Manage” section. Notice at the top right of the screen that you are at the “People” page. You can generate separate data sets for as many different users as you wish. In Manage, you will be able to administer all your data sets.

Since you have probably not yet collected much auto-log data for your student, three current users of Realize Language have made their (anonymous) data available to explore.




Select the View Dashboard icon next to the student whose data you wish to explore and you will automatically move to the Dashboard.

Overview Section

Discover new features

You can select any of the graphics to drill down further and discover more. To navigate between different data displays or to reset the time range, use the simple buttons at the top right of the screen. Take a few minutes to explore the different sections and to discover the variety of information that can become available over time as more data is collected and analyzed.

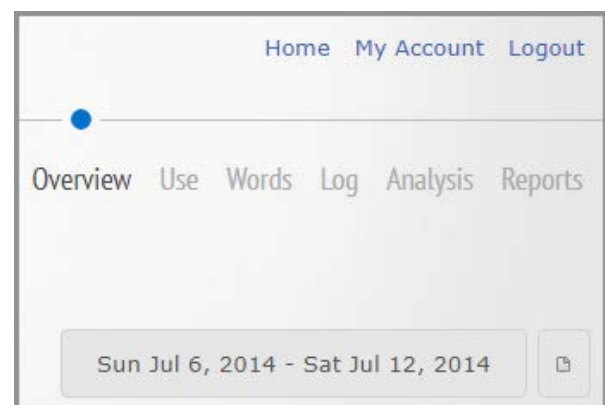

Set up a New Student

Turn on Data Logging


Realize Language is capable of analyzing and administering information from a variety of AAC devices. Refer to the manual of your system to learn how to turn “data logging” on and off. In consultation with the device user and others involved, you can collect data continuously or just at specified times.

Transfer the log file from your communication device to a USB stick in order to upload it to the Realize Language site.

Add New “People”

Realize Language leads you through the step-by-step process to add a new person’s log file.

To begin, go to **Manage** and select **Add Person**.

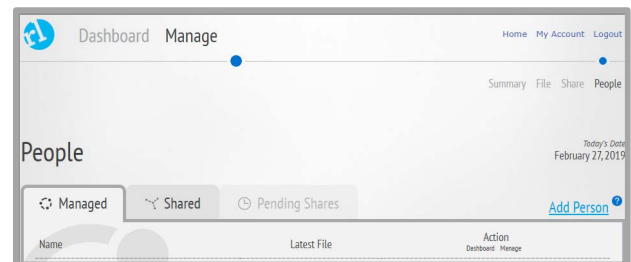

Enter the person’s name and select **Create** and then **New Source**.

In the **Add Source** window add information about the communication system and any other useful information. Now select create.


A screenshot of the 'Manage People » New' form. The form is titled 'Create a New Person'. It has a text input field for 'Name' with the value 'Adam'. Below the input field are two buttons: 'Create' and 'Cancel'.A screenshot of the 'Adam » Add Source' form. The form is titled 'Create a New Source Device'. It has a text input field for 'Name' with the value 'Accent 1000'. Below the input field is a text area for 'Description' with the value 'Direct Selection - Unity 84 Sequenced'. At the bottom of the form are two buttons: 'Create' and 'Cancel'.

Realize Language Starter's Guide



Uploading Log File to Realize Language

To begin using Realize Language all that remains is to upload the log file. Choose Select a file to upload, find the file you wish to use and then choose Upload.

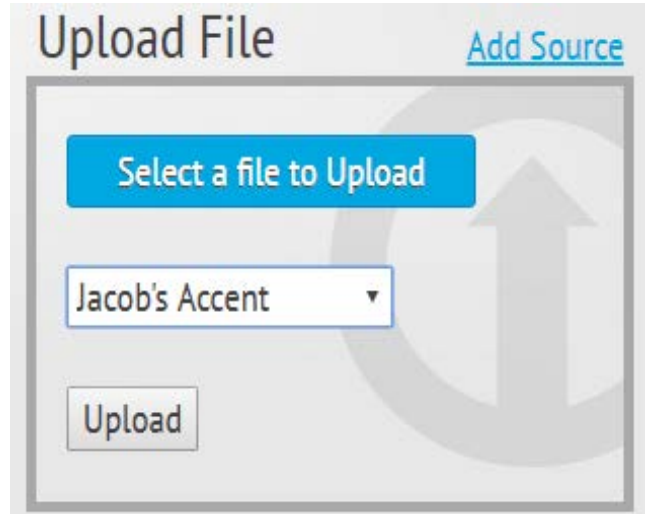

Now choose **Dashboard** and begin to explore and create reports on your AAC users language development and communication.

