

Realize Language Starter's Guide

Activity Six: Measuring minutes of use.

Select the **Use** tab.



“How much is the communication system used?” Experience shows that the amount of time that a communication device is used is quite difficult to estimate subjectively. Realize Language offers two reliable measures. On the daily scale, Realize Language measures the number of key activations in any fifteen-minute period. On the weekly and monthly scales, any fifteen-minute period containing device use counts as fifteen minutes of use.

Explore

- Note that the daily record displays the intensity of device usage on the last day of the date range in question. Click back to a previous day
- In the weekly or monthly view discover which days have been most active.

Questions

When was the busiest time of day?
Are there major variations from day to day?

Try This

- You have set a baseline to review future progress. Select the generate report button and create a full report.
- Store or print out this information as a reference baseline in the coming months.

